ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

IF YOU ARE USING THE CRYO/CUFF WITH THE AIRCAST CRYO/CUFF UNIT BE SURE YOU READ AND UNDERSTAND ALL INSTRUCTIONS LOCATED IN THE AIRCAST CRYO/CUFF IC INSTRUCTIONS FOR USE.

INTENDEE USER PROFILE: The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. User should be able to read and understand the directions, warnings and cautions.

OPERATING PRINCIPLE: The AirCast Cryo/Cuff provides cold therapy and compression.

INTENDEE USE/INDICATIONS: The AirCast Cryo/Cuff combines focal compression with cold to minimize swelling and pain.

FREQUENCY OF USE: A licensed health care practitioner must select the length of use and/or frequency of use depending on individual patients needs. Do NOT use this device if a prescription has not been provided to you or if you do not understand the prescription as this may result in injury. This device can be cold enough to cause serious injury be sure to check skin frequently.

CONTRAINDICATIONS: Cryotherapy should not be used on persons with Raynaud's or other vasospastic disease, cold hypersensitivity, decreased circulation to the local area, diabetes, arterial disease, arterial occlusion, arteriosclerosis, cryoglobulinemia, hypercoagulable clotting disorders or other peripheral vascular diseases causing ischemia or poor local circulation, local tissue infection, cold allergy, cold hypersensitivity, previous cold injury and paronychia or hemoglobinuria.

WARNINGS AND CAUTIONS: • The device can be cold enough to cause serious injury, including tissue necrosis. • User must be able to check your skin condition under the Cuff (DO NOT use if you cannot check your skin condition frequently). People are sensitive to cold in diverse ways and may react differently to cold treatment. • Beware of burning, numbness, tingling, increased redness, discoloration, itching, increased swelling, blisters, wyes, irritation or other changes in skin condition under the Cuff or around the treatment area. If you experience any of these conditions, immediately discontinue use of the device and contact your physician. • Patient skin condition checks must occur every 1-2 hours on a routine basis. Caution must be taken by routinely monitoring the patient during use with children, elderly, incapacitated patients, severe cardiovascular diseases, those with decreased sensation, those with poor peripheral circulation, diabetes, arterial disease, arterial occlusion, arteriosclerosis, cryoglobulinemia, hypercoagulable clotting disorders or other peripheral vascular diseases causing ischemia or poor local circulation, local tissue infection, cold allergy, cold hypersensitivity, previous cold injury and paronychia or hemoglobinuria.

• It is recommended that at least 20 minutes of cessation should be encouraged between 2 hour long cryotherapy sessions. • When applying the device wrap to the skin, apply a barrier (e.g. cloth or bandage) between the wrap and your skin, and do not wrap it around the area tightly. No part of the wrap should touch the skin for any period of time. • Check for moisture. If moisture is present on the Cuff, immediately discontinue use of the device. • Do not use an elastic wrap in conjunction with the Cryo/Cuff. • Dressings used under the Cryo/Cuff should be applied lightly. • When filling the Cryo/Cuff, do not raise the cooler higher than 15 inches (38 cm) above the Cuff to avoid excessive pressure. • Do not use pressure with any sense of discomfort, numbness or tingling of the limb. • The Cryo/Cuff is designed for single patient use only and may be used on the same patient for the length of treatment. • The device should be cleaned if it is used on the same patient for an extended period of time (see "Cleaning Instructions" for cleaning information). • Keep small parts and packaging materials away from small children and animals. These items pose a risk for suffocation or strangulation. • It could be unsafe to use accessories, detachable parts and materials, or connect to other equipment.

APPLICATION INFORMATION:

- PREPARE COOLER:** Connect blue tube to cooler. Add water to line inside cooler, then add ice to top of cooler. Lay the insulation disk on top of it. Attach cooler lid snugly. Allow 5 minutes with occasional shaking to chill water.
- APPLY THE EMPTY CUFF:** Unfasten posterior strap and open cuff flat. Position cuff over foot with logo facing up and quick-disconnect pointing toward toes (Fig 1). Wrap ends of cuff under foot and secure together. Wrap posterior strap around the back of the heel and secure to side of cuff (Fig 2).
- FILL CUFF:** Connect blue tube to the cuff. Listen for a click/snap. When using the Gravity Cryo/Cuff (not motorized), open air vent on cooler lid and raise cooler more than 15 inches (38 cm) above the cuff (Fig 3). If the cooler is raised more than 15 inches (38 cm) the cuff will increase in volume and weight. Hold raised cooler for about 30 seconds while cuff fills. Close air vent on cooler lid. The cooler can now be disconnected from cuff pressing the metal tab on the quick-disconnect while cooler is raised.
- RECHILL WATER:** Reconnect the blue tube to cuff, lower the cooler below cuff, and the warmed water will drain from the cuff into the cooler. Allow a minute or two for the water to mix with the ice and rechill, then raise the cooler and repeat the filling process (see step 3).

CLEANING INSTRUCTIONS: After use, completely drain water from cuff, tube, and cooler. Do drain tube, elevate tube above level of mangito. Store cooler with top off to allow drying. Periodically clean cuff, tube, and colorising a few ounces of liquid soap added to hot water. Recycle soap/water mixture between cooler and cuff a few times, then repeat with warm water only. Rinse completely.

MATERIAL CONTENTS: Polyurethane Nylon, Nylon, Polyether foam.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

EXPLANATION OF SYMBOLS:

⚠ Warnings

RX ONLY.
INTENDED FOR SINGLE PATIENT USE.

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